

# COMPRESIV

## JUMBO/JUMBO PLUS

### SCREW HOLE SEQUENCE

Example shown with 3.5  
Jumbo Plate Item #: XC35JR



**1 CORTICAL SCREW**  
*Partially tightened*

**2 LOCKING SCREW**  
*Fully tighten*

**3 LOCKING SCREW**  
*Fully tighten*

**4 LOCKING SCREW**  
*Fully tighten*

**5 CORTICAL SCREW**  
Compression Slot  
*Fully tighten*

**6 LOCKING SCREW**  
*Fully tighten*

**7 LOCKING SCREW**  
*Fully tighten*

**LOCKING SCREW** **8**  
*Fully tighten*

**LOCKING SCREW** **9**  
*Fully tighten*

**LOCKING SCREW** **10**  
*Fully tighten*

**LOCKING SCREW** **10**  
*Fully tighten*

Note: The screw sequence is a recommended guide only.  
In certain situations, this exact sequence may not be able to be executed.

©2024 Movora



# COMPRESIV BROAD

## SCREW HOLE SEQUENCE



1

**CORTICAL SCREW**  
Pivot point in Load Position  
*Not fully tightened*

2

**LOCKING SCREW**  
*Fully tighten*

3

**LOCKING SCREW**  
*Fully tighten*

4

**LOCKING SCREW**  
*Fully tighten*

5

**CORTICAL SCREW**  
Cranial Compression Slot  
*Fully tighten in Load Position*

6

**CORTICAL SCREW**  
*Fully tighten for Axial Compression*

Use a  
STA-FIX pin  
distally for  
temporary  
fixation

7

**LOCKING SCREW**  
*Fully tighten*

8

**LOCKING SCREW**  
*Fully tighten*

9

**LOCKING SCREW**  
*Fully tighten*

Note: The screw sequence is a recommended guide only.  
In certain situations, this exact sequence may not be able to be executed.

©2024 Movora