



# VERSIV JUMBO/JUMBO PLUS

## SCREW HOLE SEQUENCE

Example shown with 3.5  
Jumbo Plate Item #: ETC-35JR

**1 CORTICAL SCREW**  
*Partially tightened*

**2 LOCKING SCREW**  
*Fully tighten*

**3 LOCKING SCREW**  
*Fully tighten*

**4 LOCKING SCREW**  
*Fully tighten*

**5 CORTICAL SCREW**  
Compression Slot  
*Fully tighten*

**6 LOCKING or CORTICAL SCREW**  
For Additional Compression  
*Fully tighten*

**7 LOCKING SCREW**  
*Fully tighten*



Use a  
STA-FIX pin  
distally for  
temporary  
fixation

**8 LOCKING SCREW**  
*Fully tighten*

**9 LOCKING SCREW**  
*Fully tighten*

**10 LOCKING SCREW**  
*Fully tighten*

**11 LOCKING SCREW**  
*Fully tighten*

Note: The screw sequence is a recommended guide only.  
In certain situations, this exact sequence may not be able to be executed.

©2024 Movera

# VERSIV 3.5 BROAD

## SCREW HOLE SEQUENCE

**1 CORTICAL SCREW**  
*Partially tightened*

**2 LOCKING SCREW**  
*Fully tighten*

**3 LOCKING SCREW**  
*Fully tighten*

**4 LOCKING SCREW**  
*Fully tighten*

**5 CORTICAL SCREW**  
Compression Slot  
*Fully tighten*

**6 LOCKING or CORTICAL SCREW**  
For Additional Compression  
*Fully tighten*

**LOCKING SCREW** **7**  
*Fully tighten*

**LOCKING SCREW** **8**  
*Fully tighten*

**LOCKING SCREW** **9**  
*Fully tighten*

Use a  
STA-FIX pin  
distally for  
temporary  
fixation



Note: The screw sequence is a recommended guide only.  
In certain situations, this exact sequence may not be able to be executed.

©2024 Movora